

carilex®

DUAL GT

Dual GT is a powerful deluxe mattress replacement system packed with intuitive design to create maximum comfort while providing exceptional protection with an Anti-Bottoming Out Sensor and interchangeable air cells for total and constant pressure redistribution.

A Pressure Indication

The Visual indicator demonstrates the optimal pressure level for the patient

B Intelligent Pressure Sensing

Respond to patient movement on the mattress by automatically adjusting internal mattress pressure to regulate airflow and provide envelopment for patients

C Multiple Cycle Time

Provides a choice of multiple alternating pressure operating cycles

D Comfort Setting

Comfort adjustment options provide the optimal pressure according to each patient's sensation

E Max Inflate

Using the inflation button of the system will rapidly bring the mattress to maximum steady pressure, which allows the caregiver to perform the daily nursing procedures

F Dynamic / Static Therapy Modes

With dynamic and static mode selections to meet individual patients' needs

Memory Recall

Automatically restore to the previous settings after abnormal power failure



| | | | | | | | | | |
|--|------------------------------|--|-------------------------|--|----------------------|--|-------------------------|--|--------------------|
| | Alternating Pressure Relief | | Pressure Redistribution | | Patient Envelopment | | Memory Recall | | Low Pressure Alarm |
| | High Pressure Alarm | | Power Failure Alarm | | Alarm Mute | | Whisper Quiet | | MAX Inflation |
| | Panel Lock | | Lightweight Power Unit | | Mattress Replacement | | Cable Management | | Dartex® Coverlet |
| | Anti-Bottoming Sacrum Design | | TPU Soft Air Cells | | Protective PE layer | | Multiple Mattress Sizes | | Directional snap |
| | Durable Material | | Quick Connect Coupling | | CPR | | Easy Cleaning | | |

Other Features

THE DUAL GT MATTRESS

DualGT mattress air cells have a modular structure, each independent and divided into a different zone but fully interconnected. The head zone is always at constant low pressure; Torso and leg zone support respectively the chest and the legs area and provide extra protection for the Scrum area, which is often one of the most delicate areas of the patient and require the most attention.

1 Easily Exchangeable Air Cells / Bridging Cells

Replaceable design with 5" cell choice to provide constant pressure relief

2 The Anti-Bottoming Out Sensor

Extra protection for patients in the sacrum area

3 Instant CPR

Rapid delating the entire mattress within seconds in case of emergency

4 Square Air Cell

No gap between air cells delivers maximum support and pressure redistribution while a patient is lying down and also give additional safety for entering and exiting the mattress



Excellent Hygiene Control

Durable and washable coverlet that completely covers the mattress. Hospital-grade mattress base with easy wipe clean material.

Easy Carry Design / Color Code

Color-coded straps are easily recognizable even when the mattress is folded.

| | | |
|----------------|------------------------------------|--|
| Technical Data | Power Unit Model | REF : SR361 |
| | Power Input | 220-240Vac 50 / 60Hz 0.2A |
| | Power Consumption | Max : 15W |
| | Weight | <ul style="list-style-type: none"> Power Unit : 2.8Kg (6.2lb) Mattress Set : 10kg, 12.5kg (22lb, 28lb) |
| | Dimension | Power Unit : 12" x 9" x 5" (29.5 x 22.5 x 12cm) (W x H x D) Mattress Set (W x L x H) : <ul style="list-style-type: none"> 32" x 80" x 8" (82 x 200 x 21cm) 36" x 80" x 8" (90 x 200 x 21cm) Other size upon request |
| | Agency Approval | <ul style="list-style-type: none"> IEC/EN 60601-1 ANSI AAMI ES60601-1 AS/NZS IEC60601.1 |
| | Operating Condition | <ul style="list-style-type: none"> Temperature : 5°C to 40°C (41°F to 104°F) Humidity : 15% to 90% |
| | Transportation & Storage Condition | <ul style="list-style-type: none"> Temperature : -25°C to 70°C (-13°F to 158°F) Humidity : 0% to 90% |
| | Weight Capacity | 250kg (550lb) |

This Power Unit is not AP/APG protected

www.carilexmedical.com

Carilex Medical, Inc. **HQ**

TEL : +886-3-328-7882
 Email : amea@carilexmedical.com
 No.77, Keji 1st Rd., Guishan District,
 Taoyuan City (33383), Taiwan

Carilex Medical USA

TEL : +1-866-551-4227
 Email : americas@carilexmedicalusa.com
 17068 Evergreen Place, Unit A, City of Industry,
 CA 91745 USA

Carilex Medical DE

TEL : +49-4121-476896
 Email : eu@carilexmedical.com
 Fritz-Strassmann Str. 5, 25337 Elmshorn
 Germany

Carilex Medical NL

TEL : +31-2089-05788
 Email : nl@carilexmedical.com
 Overschiestraat 59, 1062XD
 Amsterdam

Carilex Medical UK

TEL : +44-1323-761900
 Email : uk@carilexmedical.com
 5th Floor, 167-169 Great Portland St,
 London W1W 5PF, United Kingdom